


## Bottom Buddy Toilet Aid

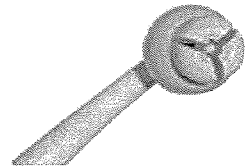

Sometimes you need assistance reaching hard to reach areas on your body and reaching after using the toilet is one of the most sensitive. There are a number of conditions that can make it difficult to reach and properly use toilet tissue. Arthritis, injury and any of a number of other conditions can make it difficult or impossible to reach without unnecessarily stressing your back or shoulders. The Bottom Buddy™ is an award winning, ergonomically designed toilet aid that makes it easier.

The 11" long Bottom Buddy has a soft, flexible head with 3 tulip-petal sections. You use your finger to gently press any toilet tissue or pre-moistened wipe into the head where it will be gripped securely. Once inserted, the toilet tissue covers the rounded head. When you are finished, sliding the button forward pushes the used tissue into the toilet bowl. No sticking, no touching, no mess. Includes a carry pouch so you can easily take your Bottom Buddy with you when you travel.




Features of the Bottom Buddy include:

- ◆ **Sturdy Construction** - the Bottom Buddy is molded using high grade, high impact plastic parts designed for long use.
- ◆ **Ergonomic Design** - The Bottom Buddy's curved shape is designed to allow you to apply gentle pressure for maximum effective cleaning.
- ◆ **Easy to Load** - The Bottom Buddy is easy to prepare for use:

1. Hold tissue with finger tip in center	2. Put finger tip into center of gripper	3. Press tissue into gripper	4. Smooth tissue over head

- ◆ **Easy to Clear** - When you're finished, drop the tissue into the bowl:

*Point head down and slide button forward to release tissue*

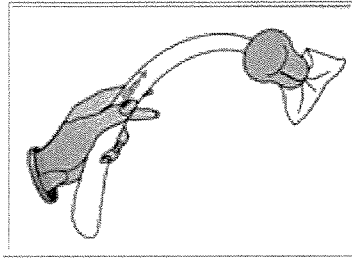

◆ **Choice of Wipes** - the gripper can securely grip toilet paper, pre-moistened wipes or any other similar materials.

If you need a little help reaching after using the toilet the Bottom Buddy will make it easier.